

PRESSALIT.

Product	VALUE support arm, height adjustable	
Item number	R1110000	
	VVS:615722405	
	NRF:6191643	
	RSK:7972412	
	EAN:5708590176420	
	HMI: 25413	
	NBS:N13:436	
	FDA:ILS/D135866	
Colour	White	
Δdiustahility	The ergonomic design gives a secure and comfortable grip as well as a good sen of where the support arm ends. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately. VALUE support arm, wall-mounted, foldable, height adjustable 150 mm, projection 865 mm. The ergonomic design gives a secure and comfortable grip as well as a good sen of where the support arm ends. Materials: Rilsan-coated steel, diecast aluminium, plastic. Incl. fixing bolts for brick wall. Plugs depend on wall type and must be ordered separately.	
Adjustability	Height adjustable	
Adjustment height	150 mm	
Product capacity	Maximum load: 120 kg. When the support arm is loaded with 120 kg, each wall fastener is loaded with 86 kg	
Materials	The support arm is manufactured from steel with a Rilsan coating, which is comfortable to the touch, easy to clean, and protects the metal against corrosion. The end caps are manufactured from nylon (PA).	



PRESSALIT.

	T		
Product	VALUE support arm, height adjustable R1110000		
Item number			
Ergonomy	Provides firm support when you are sitting, standing, or transferring to and from a toilet.		
	Good, broad support surface for the hand and arm. The rounded form gives a firm and comfortable grip.		
Features	The support arm can fold up when not in use or to make more space.		
	Height adjustable 150 mm.		
Warranty	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.		
Cleaning	VALUE support arms. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged. For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products. The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container. DISINFECTANTS: Clinimax Difficil-S Erisan OXY+ Klorilli		
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200		
	The product must be wiped down with a soft rag wrung out in clear water immediat after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.		
Declaration of conformity	CE MDR This product is in compliance with all relevant requirements of: - Regulation (EU) 2017/745 on medical devices.		



PRESSALIT.

VALUE support arm, height adjustable

Item number

R1110000

Installation in different wall types

a. V. O. C	
C. C. C. C. V. V	
2 · D · D · O · D · O	

Concrete >B25

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.






Solid Brick >= Mz12

V8592 - 5 countersunk screws \emptyset 10 x 80, with white screw covers are included.

Plugs to be ordered separately.



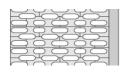

Solid Sand Lime Brick >= Hlz12

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.






Aerated Concrete P4

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.



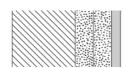

Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood

V8592 - 5 countersunk screws Ø10 x 80, with white screw covers are included.

Plugs to be ordered separately.





PRESSALIT®

Product Item number	VALUE support arm, height adjustable R1110000	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm) Wall fasteners (5 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant	
	Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.

pressalit.com

Pressalit A/S
Pressalitvej 1
DK-8680 Ry
Tel.: +45 8788 8788
Fax: +45 8788 8789
E-mail: pressalit@pressalit.com